

dermalogica professional treatments in a medical setting

dermalogica treatments with the bt-nano™ machine

When combining professional Dermalogica treatments and microcurrent with the bt-nano™ microcurrent, it is important to:

1. Review the Consultation Card to provide an in-depth consultation of your patient/client's skin ensuring they are a candidate for treatment.
2. Ensure the skin is in optimum health to reduce possible complications.
3. Never leave your client unattended when using an electrical modality.
4. Use Dermalogica professional products that are suitable for the patient/client's current skin condition.
5. Directions and protocols may differ from machine to machine; always consult the manufacturer for exact protocols and product usage for your particular microcurrent machine. Microcurrent can be used for muscle re-education or specific iontophoresis.

Definitions with microcurrent therapy:

- **Preparation 1 and 2** – Muscle re-education
- **Skin 1, 2 and 3** – Specific Iontophoresis (product penetration)

Dermalogica treatment product recommendations based on skin condition for use with bt-nano™ Microcurrent therapy:

Premature ageing skin:

- Cleanse with **PreCleanse** followed by **Skin Resurfacing Cleanser**
- Perform **Face Mapping®** skin analysis
- Exfoliate with **MultiVitamin Power Exfoliant** or **Exfoliant Accelerator 35**
- Remove with **Special Cleansing Gel**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Antioxidant HydraMist**
- Skin 1 – **AGE Smart® Complex**
- Skin 2 – **MultiVitamin Power Recovery® Masque** (remove after Skin 2)
- Skin 3 – **Intensive Moisture Balance** or **Power Rich™**
- Apply **Dynamic Skin Recovery SPF50** or **Solar Defense Booster SPF50**

Hyperpigmentation:

- Cleanse with **PreCleanse** followed by **Skin Resurfacing Cleanser**
- Perform **Face Mapping®** skin analysis
- Exfoliate with **Daily Microfoliant®** and **Exfoliant Accelerator 35**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Antioxidant HydraMist**
- Skin 1 – **C-12 Pure Bright Serum** with **Purifying Botanical Mixer**
- Skin 2 – **MultiVitamin Power Recovery® Masque** (remove after Skin 2)
- Skin 3 – Skin appropriate complex and **Skin Smoothing Cream**
- Apply **Pure Light SPF50**

dermalogica®

dermalogica professional treatments in a medical setting

Sensitised skin:

- Cleanse with **PreCleanse** followed by **UltraCalming™ Cleanser**
- Perform **Face Mapping®** skin analysis
- Exfoliate with **Daily Microfoliant®**
- Prep 1 – **UltraCalming™ Mist**
- Prep 2 – **UltraCalming™ Mist**
- Skin 1 – **UltraCalming™ Complex**
- Skin 2 – **Colloidal Masque Base** (remove after Skin 2)
- Skin 3 – **UltraCalming™ Serum Concentrate**
- Apply **Super Sensitive Shield SPF30** or **Ultra Sensitive Tint SPF30**

Oily/acne skin:

- Cleanse with **PreCleanse** followed by **Dermal Clay Cleanser** or **Clearing Skin Wash**
- Perform **Face Mapping®** skin analysis
- Exfoliate with **Skin Exfoliant System**
- Prep 1 – **Multi-Active Toner**
- Prep 2 – **Multi-Active Toner**
- Skin 1 – **MediBac Clearing® Complex**
- Skin 2 – **Colloidal Masque Base** (remove after Skin 2)
- Skin 3 – **Active Moist**
- Apply **Oil Free Matte SPF 30** or **Solar Defense Booster SPF50**